CRYOS CARD

MANUAL

User instructions

New Age Italia srl

Via De Brozzi, 3 - 48022 Lugo (RA)

Phone: +39-0545.32019 - Fax: +39-0545.369028

Website: www.newageitalia.it - E-mail: info@newageitalia.it

Copyright New Age Italia srl. All rights reserved. Copying or reproduction of this document by any means, including the total or partial photocopying of its content, prohibited unless with the express written consent of New Age Italia srl.

INDEX

Chap.1 – INTRODUCTION	5
1.1 – What is CRYOS CARD?	5
1.2 – Why choose the CRYOS CARD?	5
1.3 – Who is the CRYOS CARD designed for?	6
Chap.2 – INDICATIONS AND COUNTER-INDICATIONS	6
2.1 – Indications	6
2.2 – Counter-indications	6
Chap.3 – OPERATION	7
3.1 – Buttons on the CRYOS CARD	7
3.2 – Connecting and using the applicators	7
3.3 – Switching the unit on	7
3.4 – Starting treatment	8
3.5 – Setting the treatment temperature	8
3.6 – Ending the treatment	8
3.7 – Switching the unit off	8
3.8 – Symbols on the display	9
Chap.4 – PROGRAMMES	9
4.1 – Setting programmes	9
Chap.5 – APPLICATIONS	9
5.1 – Setting the treatment temperature and duration	9
5.2 – Correct body position for treatment	10
Chap.6 – POWER SUPPLY	10
6.1 – Charge level indicator	10
6.2 – Recharging	11
6.3 – Battery maintenance	11
6.4 – Replacing the battery	11
6.5 – Precautions for use	11
Chap.7 – SYMBOLS	12
Chap.8 – MAINTENANCE	13
8.1 – Applicator head	13
8.2 – Unit and power adapter	13
8.3 – Replacing the power adapter	13
8.4 – Urgent repairs:	13
Chap.9 – WARNINGS	14
Chap.10 – TECHNICAL SPECIFICATIONS	15

10.1 – Power supply	15
10.2 – Output specifications.....	15
10.3 – Other specifications	15
Chap.11 – STANDARD EQUIPMENT AND ACCESSORIES	16
11.1 – Standard equipment.....	16
11.2 – Accessories and consumable materials.....	16
Chap.12 – LITERATURE	16
Chap.13 – TABLES REQUIRED BY CEI EN 60601-1-2:2003 REGULATIONS	17

**IMPORTANT: READ THE USER MANUAL
THOROUGHLY BEFORE USE**

Chap.1 – INTRODUCTION

CRYOS CARD is part of a new range of electromedical equipment for physiotherapy. These innovative products are particularly compact, easy to use and versatile.

1.1 – What is CRYOS CARD?

Ongoing research in medical apparatus has created a new system for Cryotherapy and Thermotherapy, used in the CRYOS CARD generator. CRYOS CARD is a modern medical device for cold (cryotherapy) and/or heat (thermotherapy) treatment.

The innovative system of the CARD contains a free memory space to set and to record new protocols with personalized parameters for the patient, that can be modified or cancelled when needed.

The device is equipped with a refrigerating head fitted with a conical tip for cryotherapy. The same head can also generate heat, allowing its use for thermotherapy without necessitating magnetotherapy or laser therapy.

Cryotherapy employs very low temperatures for their anaesthetic (pain treatment), haemostatic and anti-inflammatory effects.

Thermotherapy is a treatment method using heat. This method uses devices to locally raise the temperature in the areas treated, causing vasodilatation, promoting vascularisation, increasing the quantity of oxygen transported to tissue, accelerating the metabolism and, as a result, speeding up the elimination of catabolic waste products.

The device uses a thermo-electric module (Peltier element) that generates metered, controlled levels of heat or cold, to ensure:

- Precise, constant temperatures during the entire session
- The correct duration of the treatment to achieve the desired neuromuscular, vascular or metabolic results.

Programmability, technological innovation and ease of use make this an extremely versatile, pioneering electromedical product.

1.2 – Why choose the CRYOS CARD?

The CRYOS CARD generates both cold and heat, covering the full range of temperatures generally used in physiotherapy. CRYOS CARD comes with no preset programmes, as it is intended for use by medical personnel or physiotherapists who will select the most appropriate temperature (shown on the display) and treatment duration (also shown on the display) on the basis of their own expertise. CRYOS CARD is produced with no predetermined programmes, as it is the doctor or physiotherapist who decides the temperature and duration of the therapy to suit the pathology and the extent of the problem. However, this manual provides guidelines

on how to set the optimum temperature and duration for the treatment applied by the instrument through its special head with aluminium contact tip.

1.3 – Who is the CRYOS CARD designed for?

CRYOS CARD is specifically designed for application in medicine and physiotherapy in particular. However, its great simplicity means that not only doctors and rehabilitation therapists can use it.

Chap.2 – INDICATIONS AND COUNTER-INDICATIONS

This chapter describes the main situations in which the use of magnetic fields is recommended and the limitations for this type of therapy.

2.1 – Indications

The most common applications for this therapy are:

- sports injury rehabilitation
- post trauma anti-inflammatory treatment

Significant analgic results are also achievable in the conditions mentioned above, through the direct action of the energy applied to sensitive nerve fibres and the capacity of the treatment to attenuate local inflammation. This therapy must only be performed, however, under the supervision of a doctor or physiotherapist and subject to professional diagnosis.

2.2 – Counter-indications

As the operating temperatures of the applicator tip are not harmful (from – 3 to + 50 °C), and the device is only intended for use under the supervision of a doctor or physiotherapist, this is a safe product with no particular counter-indications other than it must not be used near the eyeballs or near the uterus (abdomen/lumbar region) during pregnancy.

Chap.3 – OPERATION

3.1 – Buttons on the CRYOS CARD

- | | |
|---|---|
| 1 – Power adapter inlet (at the back) | 5 - (I) = ON/OFF: switches unit on /off |
| 2 – Interchangeable card | 6 - +/- (1): temperature setting |
| 3 - START: start programme | 7 - +/- (2): set channel 2 intensity |
| 4 - PAUSE STOP: stop or pause programme | 8 – Outlets for channels 1 and 2 |

NOTE: Before performing cryotherapy or thermotherapy, read this manual carefully and consult a doctor or physiotherapist.

3.2 – Connecting and using the applicators

Connect the applicator lead to the outlet socket on the unit. To insert, rotate the connector jack until the connections line up with the outlet. Once inserted, screw in completely to secure the leads. To apply heat/cold, touch the applicator on the area you wish to treat.

3.3 – Switching the unit on

Press button (I) – ON/OFF to switch on the CRYOS CARD. The greeting message is shown on the display and the selection menu is displayed.

WARNING: If the unit is switched on without a card inserted, the display will show the message "Insert Card". If a card is not inserted within 5 seconds, the unit switches off. Only use cards with the same serial number as the unit (see label on back of card and unit).

3.4 – Starting treatment

Select the temperature and duration of the treatment and press **START**. Once the unit reaches the required temperature, the treatment may be started.

NOTE: If an applicator head is not connected to the unit when **START** is pressed, the programme is interrupted and an alarm message is displayed. Insert the head and press **START** again.

3.5 – Setting the treatment temperature

To set the temperature, press the **TEMP +** button repeatedly or press and hold until the required treatment temperature is displayed. The treatment temperature setting increases in steps of 1°C from -5 to +50 °C. Increase the treatment duration by pressing the **TIME +** button. Similarly, press **TIME –** to decrease the duration setting. Interrupt the treatment if pain is felt or if the treated area becomes too hot.

3.6 – Ending the treatment

The programme automatically stops and the programme end warning buzzer sounds when the timer reaches 0. Press **PAUSE STOP** to stop the programme before it has completed. Press once to pause (press **START** to resume programme) or press twice to end the programme and return to the start menu.

3.7 – Switching the unit off

Press and hold the **(I) – ON/OFF** button for a few seconds to switch the unit off. If it is not used again after a programme has completed, the unit switches off automatically after a few minutes.

3.8 – Symbols on the display

The following are the symbols displayed in the bottom line of the screen during selection, setting and the programme execution.

- ERROR: incorrect operation

- BATTERY LEVEL: battery charge level

- USER: selection of user defined programmes stored

- MAINS: mains power

Chap.4 – PROGRAMMES

4.1 – Setting programmes

To use the unit, switch on, set the treatment duration and temperature and press start. The following are examples of user defined programmes for the most common applications:

Chap.5 – APPLICATIONS

5.1 – Setting the treatment temperature and duration

A correct temperature setting is essential for the effectiveness of a cryotherapy or thermotherapy programme. The setting varies depending on the type of programme used, the duration of the treatment and the characteristics of the patient. The following table describes the initial temperature setting – in other terms, the temperature immediately after pressing START – which may be increased while the programme is in progress to intensify the effects or decreased if excessive heat or pain is experienced.

Initial setting	Adjustment
The temperature may be varied by the user depending on the application (see following table).	The treatment temperature may be adjusted as follows: increase temperature to intensify the effects of thermotherapy or decrease temperature to intensify effects of cryotherapy

NOTE: if the temperature setting or its subsequent adjustment causes excessive heat or pain in the area treated, reduce the intensity of the treatment or interrupt if necessary.

The following table lists the recommended settings for the most common applications:

Application area	Recommended treatment duration
Fingers	15 minutes
Hands and feet	20 minutes
Limbs	25 minutes
Trunk	30 minutes

The above parameters are indicative, are applicable for both thermotherapy and cryotherapy and may vary depending on the advice of the doctor or physiotherapist.

5.2 – Correct body position for treatment

The ideal position is lying flat on the front or back, depending on the area being treated. This position must be maintained throughout the entire session to facilitate the treatment.

Chap.6 – POWER SUPPLY

CRYOS CARD may be powered by mains electricity, via the adapter supplied, or by the internal 24 V – 1.8 Ah rechargeable Ni-Mh batteries which, in normal conditions, enable an operation of approximately one hour. Battery duration depends on the type of treatment, the temperature setting and the treatment duration.

 For more detailed information about the battery and power adapter supplied as standard, see the chapter “Technical specifications”. The power adapter and the battery may only be replaced by expert personnel, using original components only.

6.1 – Charge level indicator

When the CRYOS CARD is powered by the internal battery, the BATTERY symbol "■" is displayed on the icon bar of the screen.

Legend:

■ - Battery charged □ - Battery empty

Intermediate charge levels are indicated by a partially filled BATTERY symbol "▣".

Recharge the battery as soon as the battery empty symbol is displayed. CRYOS CARD has an auto-off function to protect the battery. If the unit is not recharged once the battery is empty, a warning buzzer sounds and the unit is switched off.

6.2 – Recharging

To recharge the battery, connect the power adapter to the back of the unit and plug into the mains. Once battery charging starts, the battery symbol on the display fills and empties continuously. **Allow the unit to recharge until the battery symbol on the display is completely full (■).** Recharging takes up to approximately 2 hours. Once charging is complete, you may disconnect the power adapter from the mains and use the CRYOS CARD with internal battery power.

6.3 – Battery maintenance

The life of a rechargeable battery depends on the number of charge/empty cycles performed and on how these cycles are carried out. The following tips will help increase battery life: if the CRYOS CARD is not used frequently, recharge the battery once per month. To extend battery life, always empty completely before recharging (see empty battery symbol).

6.4 – Replacing the battery

How long the battery lasts after normal recharging is an indication of when the battery itself is exhausted. When the battery lasts noticeably less than when it was new, it must be replaced. Take the unit to an authorised **Service Centre** to replace the battery as a guarantee of safety and to ensure that the exhausted battery is disposed of correctly.

 Never swap the polarity of the battery connections, as this will cause irreparable damage to the electronic circuitry.

6.5 – Precautions for use

- (1) Do not dispose of exhausted batteries together with normal refuse. Hand exhausted batteries over to persons authorised for their correct disposal.
- (2) Do not dismantle or throw batteries into fire.
- (3) Do not short circuit poles.
- (4) Keep battery away from sparks or naked flame.
- (5) In the event of leakage of the battery contents and contact with the skin or clothing, rinse immediately with water. In the event of contact with the eyes, rinse with plenty of water and seek medical attention immediately.

Chap.7 – SYMBOLS

CLASS II DEVICE

TYPE BF DEVICE

IMPORTANT: READ THE DOCUMENTATION INCLUDED

THIS DEVICE IS MARKED IN ACCORDANCE WITH DIRECTIVE **0476** EEC
93/42.

Chap.8 – MAINTENANCE

8.1 – Applicator head

Check the connector lead occasionally for damage. Clean solenoids, cushion and mattress occasionally with a damp cloth.

8.2 – Unit and power adapter

Use a damp cloth to clean the unit and power adapter. Never use liquids as the unit and adapter are not impermeable (IP20).

8.3 – Replacing the power adapter

Check the condition of the power adapter insulation (casing and cables) before plugging into the mains. Replace the power adapter immediately even in the event of only slight damage.

8.4 – Urgent repairs:

Have the unit repaired immediately by New Age Italia or by authorised personnel if:

- the unit has been subjected to mechanical shock (e.g. serious fall);
- the unit has been seriously overheated (e.g. left near an intense source of heat);
- you suspect that liquid may have entered the casing;
- the power adapter, casing or any other part of the device is damaged, broken or missing;
- the unit functions anomalously.

 For safety reasons, do not use with accessories (e.g. applicator heads and power adapters) other than those supplied as standard.

 Maintenance must be performed and the unit must be tested for compliance with EN60601-1 safety standards for medical devices using a safety tester once per year. The operation lifespan of the device is guaranteed by the manufacturer only if maintenance is carried out regularly.

N.B.: Only have the unit tested by New Age Italia. The unit may be sent directly to the factory service lab or handed over to the dealer that sold the unit.

Technical service centre:

New Age Italia srl-Via De Brozzi, 3-48022 Lugo (RA)

Phone:+39-0545.32019-Fax: +39-0545.369028

Website: www.newageitalia.it - E-mail: sales@newageitalia.it

Chap.9 – WARNINGS

 Use the unit only with electrical systems complying with applicable safety legislation.

 The unit has a protection rating of IP20 (see chap. “Technical specifications”). Do not use near liquids, as the unit is not impermeable.

 Do not use in the immediate vicinity of mobile telephones (keep at least a few metres distance).

 Using close (e.g. 1 metre) to a shortwave or microwave therapy device may cause disturbances in the generator output.

 Do not use the CRYOS CARD and an HF surgical device simultaneously on the same patient, as this may be hazardous for the patient and the device itself.

 To ensure that the unit performs as specified, use only at ambient temperatures between 5 and 40° C and with humidity levels less than 80%. The same conditions are required for transporting and storing the unit.

 In the event of malfunction, send the unit to the manufacturer for inspection.

 Do not use near inflammable products.

 Do not use accessories other than those supplied. It is imperative that the patient is informed in advance of the normal sensations experienced during the therapy, so that the treatment may be interrupted immediately by switching off at the controls or removing the applicator tip should the patient notice any anomalies.

 If the temperature setting or its subsequent adjustment causes excessive heat or pain in the area treated, reduce the intensity of the treatment or interrupt if necessary.

 Keep away from children.

Chap.10 – TECHNICAL SPECIFICATIONS

10.1 – Power supply

Power Solve PSE65-24

Mains input: 100-245V 1.9A

Output: 24V 3.33A 80W

Internal power supply: Ni-MH 2 X 12V-1.8 Ah rechargeable battery

10.2 – Output specifications

Applicator tip temperature from -3 to 50 °C at an ambient temperature of 20°C

10.3 – Other specifications

- Dimensions: 175 mm x 165 mm x 100 mm height
- Weight: 1 Kg
- Class: II Type: BF
- Impermeability protection rating: IP20
- Safety rating for use near inflammable anaesthetic gases: not rated AP or APG
- Device suitable for continuous operation


Built in compliance with the following regulations:

EN 60601-1 (1998) – Electromedical devices: General regulations for safety

EN 60601-1 (1998) – Collateral regulation: Electromagnetic compatibility – Prescriptions and tests

EN 60601-1-4 (1997): Collateral regulation: Programmable electromedical systems

EN 980-2003 and EN 1041 – Symbols used for electromedical devices

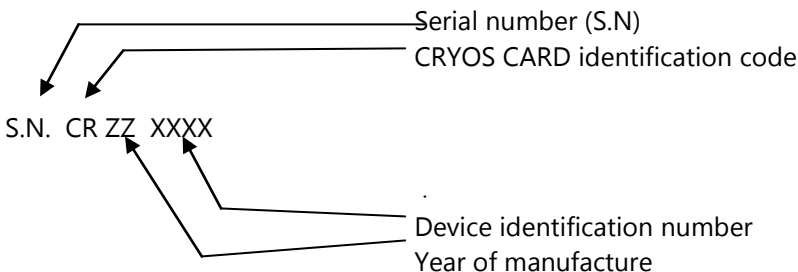Chap.11 – STANDARD EQUIPMENT AND ACCESSORIES

11.1 – Standard equipment

The CRYOS CARD comes complete with:

Unit
Medical Card
Battery charger
1 applicator tip
User manual
Carrier case

11.2 – Accessories and consumable materials

There are no additional accessories available for this unit.

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CRIOS C. is intended for use in the electromagnetic environment specified below. The customer or the user of the Model **CRIOS C.** should assure that it is used in such an environment.

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Recommended separation distances between portable and mobile RF communications equipment and the Model 006

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CRIOS C. is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the Model 006 can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the Model 006 as recommended below, according to the maximum output power of the communications equipment.

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For transmitters rated at a maximum output power not listed above, the recommended separation distance d in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

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At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

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These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.