# iTens Terapix

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# iTens Terapix

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**New Age Italia s.r.l.**

Sede Operativa: Via De Brozzi, 3 - 48022 Lugo (RA)

Tel +39 0545 32019 - Fax +39 0545 369028 - P. Iva 02299101200

**[www.newageitalia.it](http://www.newageitalia.it) - [info@newageitalia.it](mailto:info@newageitalia.it)**

# iTens Terapix

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## **IMPORTANT:**

READ THIS USER'S MANUAL THOROUGHLY BEFORE USING

# 1 INTRODUCTION

## **1.1 What is iTens Terapix?**

iTens Terapix is a new portable electrostimulator for everyday wellbeing that is particularly suitable for treating the commonest forms of pain. The memory contains 12 basic preset programmes for all types of stimulation: iTens Terapix has two independent channels for connecting up to four electrodes to stimulate two different parts of the body simultaneously. The unit can also be used on two different persons at the same time, adjusting the treatment intensity independently in accordance to each subject's sensations.

## **1.2 - Who can use iTens Terapix?**

iTens Terapix may be used by anyone who wishes to carry out specific pain relief treatments themselves at home or while travelling. It is so easy to use that treatments may be performed anywhere and at any time, to take full advantage of its beneficial effects and to achieve extraordinary results.

## **1.3 - Applications of iTens Terapix**

iTens Terapix may be used for a wide variety of applications, ranging from the analgesic treatment of joint and muscle pains, cervical and lumbar pains, tendonitis, etc. In addition to pain relief, iTens Terapix may also be used for decontracting sessions and to obtain a relaxing effect.

### 2.1 General guidelines

Before using the device for the first time, read this user's manual thoroughly. Furthermore, before switching the device on or connecting to a patient, always perform the following checks:

- (1) Check the unit visually for **signs of damage**;
- (2) Read and make note of all **warnings** printed on the unit;
- (3) Keep unit away from any **spillage hazards**, as the unit is not impermeable (IP 20);
- (4) Check that you have the **right accessories necessary** for the intended stimulation treatment (see Chap. Standard Equipment).

### 2.2 Counter-indications

The electro-stimulator may not be used in the following cases:

- (1) On persons wearing pacemakers or suffering from heart disease or arrhythmia;
- (2) On the front of the throat, as this may provoke laryngeal spasm;
- (3) On the sides of the neck, as the carotid body is particularly sensitive;
- (4) On the abdomen or in the lumbar sacral area of pregnant women;
- (5) In areas with skin lesions, mucosa, altered localised sensitivity, infection, inflammation, phlebitis or thrombophlebitis;
- (6) In the case of implanted electronic devices (e.g. pacemakers) located immediately beneath the treated area;
- (7) In the proximity of tumours and in abdominal and lumbar areas in the event of liver or kidney stones;
- (8) Keep away from children.

### 2.3 Precautions for use

**Consult a doctor** and use the electro-stimulator only under medical supervision in the following instances:

1. In order to identify the pathological cause when using neuromuscular stimulation for analgesic purposes in patients with painful syndromes, and in particular if the symptoms do not subside within a period of a few days of treatment with analgesic stimulation;
2. When applying electrodes to the face;
3. When using near areas with metallic implants (prostheses, osteosynthesis materials, screws, plates, coils, etc.);
4. In patients suffering from epilepsy or Parkinson's disease, as their sensitivity threshold may be altered;
5. In the case of suspected or confirmed pregnancy and applying only to the extremities;
6. When applying electrodes to areas near tendon or muscle injuries;
7. With patients suffering from spastic paralysis.

### **Do not apply electrodes in the following areas:**

1. Above the eyeball;
2. Directly onto sensitive areas, such as the centre of the breast, the groin or the armpit.
3. The heart must never be situated between the positive and negative electrodes. Never apply an electrode on the front of the chest and one on the back near the heart.

### **2.4 Special Precautions**

1. Do not use the electro-stimulator after a **large meal**.
2. If symptoms of **tachycardia, extrasystole** or **intolerance to stimulation** arise, **suspend** treatment to restore normality.
3. After treatment, a **slight reddening** of the skin where the electrode was applied is possible; in this case, we recommend using a **lower frequency** or a **lower stimulation intensity**.
4. The electrodes are intended for personal use. Using the same electrodes or conductive straps on more than one patient may lead to cross-infection.

## 3 FUNCTIONS

### Name and function of the device

1. Output channels 1 and 2 adjustable respectively with the CH1 and CH2.

2. LCD interface: display the usage information

3-4. Keys:

**Mode:** used to select the desired program.

**Timer:** used to set the time and to select the silent function of the acoustic signal.

**Buttons CH1 and CH2:** these are the buttons for adjusting the intensity.

Use them to adjust the output intensity of channel 1 and channel 2 (levels from 0 to 40).


**ON / OFF power button:** used to turn the device on or off and to reset the level of Intensity during a treatment.

5. Belt clip


6. Battery cover

## Operation of the product


1. Turn on the device: In power-saving mode press the to enter standby mode, indicating LCD.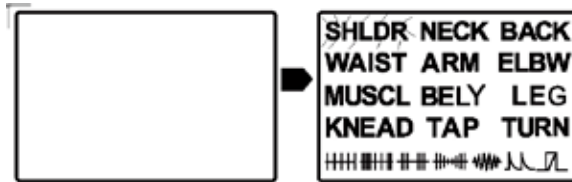


2. Mode selection: press the mode key to select the mode, and the selected will flash.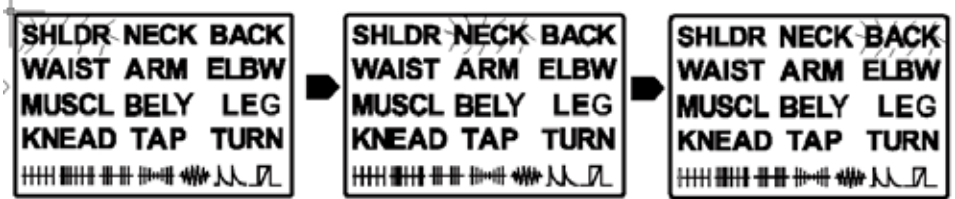

3. Setting the time: press the timer button to set the usage time. The settable time goes from 5 to 90 minutes.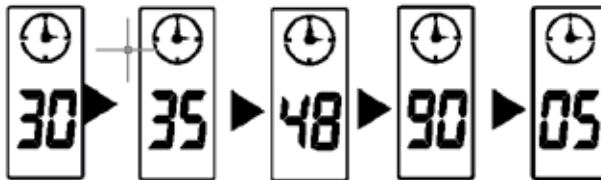


4. Start treatment or increase intensity: press [CH1+] or [CH2+] strength level 0~40, adjust 1 level in per step.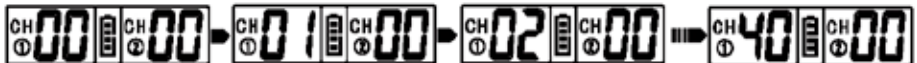

The timer will be counted down during the time of treatment, and the output will be stopped automatically after 30 minutes of massage.


5. Lower intensity: press [CH1-] or [CH2-], adjust 1 level in per step. Note: CH1 and CH2 are two independent control channels. CH1 is the output intensity of controlling CH1 channel, and CH2 is the output intensity of controlling CH2 channel.

6. End of timer: stop output at the end of timer and return to standby mode.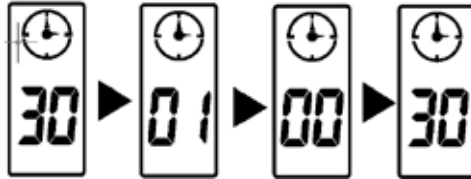


7. During the treatment, if the power button is pressed, the intensity will become 0 and the output will stop.


8. During the treatment, if no button is pressed for 15 seconds, the device goes into lock mode. To unlock the device press the CH- button of one of the two channels.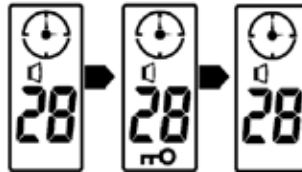


9. Switch off: press the power button, in stand-by mode, to switch off the device.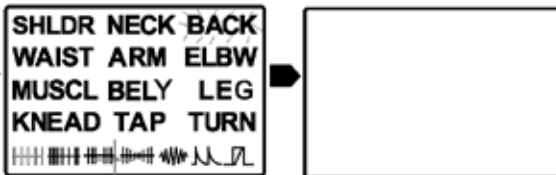


10. Mute setting: long press the key of time setting 2 seconds to switch.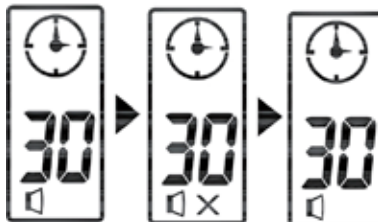

11. Safe intelligent sensing function: If the conductive adhesive fails to contact the skin normally, the output strength of the machine will be 0 so that to ensure the user's safety. If you want to continue to use it, just reattach the electrode plate to the skin and return to the original strength.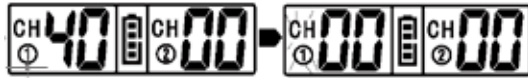

12. Automatically enters the power-saving mode: automatically enters the power-saving mode when there is no operation in the next 3 minutes.

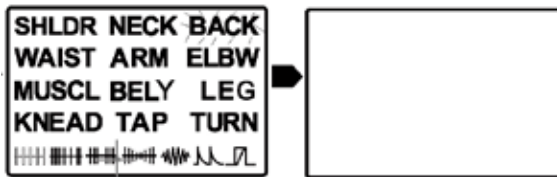**Warm tip 1:** The greater the output intensity does not represent the better the effect. The most comfortable and effective is, when the skin part feels current output, then increase 2~3 or so.

**Warm tip 2:** In the treatment process, it is forbidden to remove the electrode of the device. If you do this, you will have an uncomfortable current stimulation in your fingers, and the correct way to do it is to turn the device off and take portable plate off.

## 4 LIST OF PROGRAMS

iTens terapix has 12 programs for the most common applications:

**SHLDR** *shoulder*

**NECK** *neck*

**BACK** *back*

**WAIST** *waist*

**ARM** *arm*

**ELBW** *elbow*

**MUSCL** *muscles*

**BELY** *abdomen*

**LEG** *legs*

**KNEAD** *program hand massage*

**TAP** *program tap massage*

**TURN** *program turning massage*

### 4.1 - Programming treatment sessions

For effective treatment with iTens Terapix, apply stimulation to the painful area. For acute pathologies and particularly intense pain, the analgesic therapy may be repeated 2 or 3 times the same day at intervals of a few hours. Recommended number of weekly sessions:

- **Analgesic therapy** – every day until the pain disappears or significantly subsides (consult a doctor if the pain persists after 10/20 applications).

## 5 TECHNICAL DATA

### 5.1 Characteristics

- Power supply: 3 x 1.5 V AA batteries
- Waveform: TENS
- Channels: 2 independent
- Intensity: 0-100 mA
- Frequency: 1-120 Hz
- Pulse amplitude: 100-350  $\mu$ s

### 5.2 - Standard equipment

- Unit
- 4 electrodes pre-coated with adhesive gel
- 3 x 1.5 V AA batteries
- User's manual
- 2 cables
- Carrier case

## 6 SYMBOLS

APPLIED PART OF TYPE BF

DISPOSAL IN ACCORDANCE WITH DIRECTIVE 2002/96/EC AND 2003/108EC

REFER TO INSTRUCTION MANUAL.

CE 0123

## 7 MAINTENANCE

- **Unit:** clean with a moist cloth. To ensure that the unit performs as specified, use only at ambient temperatures between 5 and 40° C and with humidity levels less than 80%. The same conditions are required for transporting and storing the unit.
- **Batteries:** remove the batteries from the unit for prolonged periods out of use. Replace dead batteries with batteries of the same type or rechargeable batteries; do not use different types of battery together.
- **Electrodes:** in normal operating conditions, the electrodes may be reused at least 10-15 times. After use, return the electrodes immediately to their relative backings and seal in their bags; when the gel starts to lose adhesive power, wet the pre-coated side with water before and after use. Store electrodes in the refrigerator to extend usable lifetime.

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