Tekra life

●●● Your Medical System

DECLARATION OF CONFORMITY

New Age Italia Srl

Via De Brozzi, 3 – 48022 Lugo (RA)

Tel 0545/32019 – Fax 0545/369028

P. Iva 02299101200

in the person of its legal representative, Mr. Barnabè Manuel declares that
the following class II b device:

Brand Name: New Age Italia

Model: TEKRA LIFE

is a MADE IN ITALY product, made by New Age Italia s.r.l.;

is up-to the requisites prescribed in EEC Directive 93/42/EEC concerning health regulations and absorbed by Legislative Decree no. 46 of February 24th, 1997;

that the procedure for the evaluation of conformity described in Annex II to the
above-mentioned Directive has been duly performed;

and that the following harmonized standards have been applied:

EN60601-1 : 2007

EN60601-1-2 : 2001

EN60601-1-4 : 1997

EN 60601-2-10 : 2001

EN 980 : 2002

The notified authority no. 0123 has, in virtue of the certificate G2120778453003,
issued a permission to apply a CE conformity mark to the above-mentioned
regulations prescribed in EEC Directive 93/42/EEC and 2007/47/CE

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User manual
Instructions

Tekra life

1. PRESENTATION	18
1.1 What is TEKRA LIFE?	18
1.2 What is TEKRA LIFE for?	18
1.3 Who is TEKRA LIFE for?	18
2. INDICATIONS AND CONTRAINDICATIONS	19
2.1 Indications	19
2.2 Contraindications	19
3. OPERATION	20
3.1 Handpiece connection and application	21
3.2 Device power supply	21
3.3 Turn on the device	21
3.4 Program selection	21
3.5 Start of the therapy	22
3.6 Setting the output intensity	22
3.7 Setting Time	22
3.8 End or interruption of the session	22
3.9 Turn off the device	22
3.10 Symbols on display	23
4. FREE PROGRAMS	23
4.1 Setting	23
5. APPLICATIONS	24
5.1 Therapy planning	24
5.2 Position to keep during the sessions	24

Tekra life

6. POWER SUPPLY	24
6.1 Use precautions	24
7. MAINTENANCE	24
7.1 Headpiece	24
7.2 Device and feeder	24
7.3 Feeder substitution	24
7.4 Immediate maintenance	25
8. WARNINGS	26
9. TECHNICAL FEATURES	27
9.1 Power supply	27
9.2 Output features	27
9.3 Other features	27
10. SYMBOLS	28
11. ENDOWMENT OF BASE / ACCESSORIES	28
11.1 Basic Endowment	28
11.2 Accessories and consumed materials	28
12. BIBLIOGRAPHY	28

ATTENTION:
READ CAREFULLY
THE INSTRUCTIONS BEFORE USE

TEKRA LIFE belongs to the new line of electromedical instruments for physiotherapy. The reduced dimensions, the facility of use and the versatility are the principal characteristics of this innovative line of products.

1.1 - What is TEKRA LIFE?

The constant research in the field of medical devices has brought to the creation of the new system for diathermy therapy, constituted by the generator TEKRA LIFE. The innovative software inserted inside the instrument, allows the generation of Radio frequency waves for therapeutic purpose for the treatment of common pathologies (lesions, accidents, illnesses) through the use of preset programs ready to the use. The technological innovation and the facility of employment make it an **extremely versatile and innovative** product in the electromedical instruments field.

1.2 - What is TEKRA LIFE for?

With TEKRA LIFE is possible to apply radiofrequency technology with benefic effects on the reconstruction of the bones tissue and, in general, for the regeneration of damaged tissues, in order to obtain an antalgic, antinflammatory, antiedemic effect.

1.3 - Who is TEKRA LIFE for?

TEKRA LIFE finds in the home therapy the best environment to express completely its own potentialities thanks to the simplicity and versatility of the product. The reduced dimensions, moreover, allows to transport it easily.

ATTENTION

For home use, please follow your doctor's indications.

2

INDICATIONS AND CONTRAINDICATIONS

2.1 - Indications

The most common pathologies that healed with radiofrequency are those up to the apparatus osteo-muscle-tendineo. They are particularly suitable for the treatment of:

- Inflammatory and degenerative arthropathies;
- edemas;
- wrinkles;
- muscular contractures;
- cellulite;
- stretch marks.

We always suggest to ask and follow your doctor's instructions for diagnosis and therapy application.

2.2 - Contraindications

The Radiofrequency waves have a few counter indications common to all the devices which works with endogenic production of heat.

- presence of metallic fragments;
- pregnancy;
- tumors;
- tuberculosis;
- acute inflammatory process;
- cutaneous lesions;
- pace makers;
- alterations of the sensibility.

3 OPERATION

1	Display LCD
2	ON/OFF, START/PAUSE
3	Cursor up/down
4	(+)/(-) : intensity regulation
5	Output for radio frequency handpiece cable
6	Power supply

NOTE

Before applying the radiofrequency, check the side effects and consult a doctor.

3.1 - Handpiece connection and application

Connect the cable of the handpiece to the output socket of exit of the instrument. Insert the connectors and rotate until the connections coincide with the output socket; once inserted, screw it until the end to fix firmly the cable to the instrument. Always use the handpiece with the cream in endowment.

3.2 - Device power supply

TEKRA LIFE must be used connecting the device to the power supplier and a regular electricity plug.

3.3 - Turn on the device

To turn on TEKRA LIFE press the key (2) - **ON/OFF**.

3.4 - Program selection

Turn on the device and find the text PROG. Choose the desired program using the keys (+)/(-)

The preset programs are 15 and are indicated by the letter "P" (see programs table). The Free programs are 5 and are indicated by the letter "U".

Programs Table

NUM.	PROGRAM
P1	Hand arthritis
P2	Headache
P3	Rhinitis
P4	Sinusitis
P5	Contractures
P6	Edema
P7	Chronic pain
P8	Acute pain
P9	Acne rosacea
P10	Acne conglomerate
P11	Face wrinkles
P12	Neck wrinkles
P13	Face lifting
P14	Celulitis
P15	Strecth marks

NUM.	PROGRAM
U1	
U2	
U3	
U4	
U5	

3.5 - Start of the therapy

When the display shows the requested program press START to begin the therapy; Once pressed START, the display will show the remaining time.

3.6 -Setting the output intensity

Intensity is already set but can be adjusted manually. Go to INTEN menu using the keys ↑ and ↓ and change the parameter according to patient's needs using (+)/(-) keys.

NOTE: in case of pain or excessive hot, turn down the temperature pressing key (-) or interrupt the therapy pressing PAUSE.

3.7 - Setting Time

It's possible increase a program duration (Timer) to have a long therapy for some hours or for night applications, up to eight hours. To adjust the Timer during operation of a program use the keys ↑ and ↓ to go to the TIME menu, then change the parameter with (+)/(-). Each push increase or decrease for 5 minutes.

3.8 - End or interruption of the session

If you want to interrupt before the end press the key **(2) PAUSE**: once for a break (to continue press **START**); twice to finish the program.

3.9 - Turn off the device

To turn off the instrument press and hold the key **(2) ON/OFF** for some seconds. If after the end of a program the instrument is not used, it automatically turns off after a few minutes.

3.10 - Symbols on display

MAIN FEEDING: power supply through electrical power (charger connected).

4

FREE PROGRAMS

4.1 - Setting

To access the setting of free programs from PROG menu area it is necessary to use the keys (+) / (-) and go to the list of programs starting with U. Confirm and with \uparrow / \downarrow go to INTEN, FREQ and TIME menu and change the parameters as desired with the keys (+)/(-). Once the parameters have been selected press key **2 START** to start the selected therapy. When the device will be turned off the program will be saved.

NAME ON DISPLAY	PARAMETER	AVAILABLE VALUES
Inten	Preset intensity	1-100%
Time	Program Duration	5-30 min

To set correctly the parameters in the free protocols, we give you a few guide lines in the following chart: in the first column you'll find the available parameters with the respective setting values, while in the second column we list the areas of affiliation and the relative type of applications.

Intensity	Effect
1-30	Relaxing/Antistress
30-60	Vascular/Altered functions
60-80	Analgesic
80-100	Heating
Program Duration (Time)	Target
Up to 10 min	Analgesic
20 min	Regenerating effect
30 min	Relaxing

5

APPLICATIONS

5.1 - Therapy planning

Application should be made every day, with the most suitable program to your pathology, until the symptoms disappear or are sensibly reduced. **At the end of the therapy, if the problem is still present, consult your physician.**

5.2 - Position to keep during the sessions

The ideal position is the most relaxed one, with the body extended supine or prone according to the zone of application. The position must be maintained during all the session.

6

POWER SUPPLY

TEKRA LIFE must be supplied by power mains with the feeder in endowment.

6.1 - Use precautions

- (1) not to short-circuit the terminals.
- (2) to avoid to provoke sparks or flames, above or around the battery.

7

MAINTENANCE

7.1 - Headpiece

The cable of connection must be periodically checked to verify the absence of cracks which are possible cause of electric dispersion; periodically clean the handpiece with a damp cloth.

7.2 - Device and feeder

To clean both the feeder and the instrument, it's recommended to use a damp cloth. Don't use in any case liquid, because these devices **are not protected from their entry (IP20)**.

7.3 - Feeder substitution

Check the state of the isolations (wrap and cables) of the feeder, before connecting it. If it's damaged, even if partially, immediately replace the feeder.

7.4 - Immediate maintenance

Immediate maintenance must be executed by New Age Italia or authorized personnel if one of the following situations happens:

- external mechanical solicitations (ex. serious falls);
- strong overheating (ex. if left next to sources of intense heat);
- some liquids can be penetrated inside the device;
- the feeder, the wrap or other parts of the instrument are damaged, broken or lacking;
- the functionality of the instrument appears altered.

For safety we recommend to never operate with accessories (es. headpiece and feeder) different from the furnished ones as endowment of base.

The frequency of maintenance, of functional control and verification of correspondence to the safety norms EN60601-1 for the medical devices should be performed with secur-tester annually. The useful life of the tool is guaranteed only from the firm if such maintenance is regularly effected.

PLEASE NOTE

It is recommended to have the controls performed annually only by New Age Italia; the device can be sent directly to the laboratories of assistance or delivered to the retailer where it has been purchased.

Assistance:

New Age Italia srl

Via De Brozzi, 3 - 48022 Lugo (RA)

Tel 0545/32019 - Fax 0545/369028

asstecnica@newageitalia.it

8

WARNINGS

-  Pay a particular attention in the employment of the headpiece not to compromise the effectiveness of the treatment.
-  Use the instrument only with electric structures conforming to the actual safety Norms.
-  The instrument has degree of protection IP20 (see Chap. "Technical features") and do not use it close to liquids, because it's not protected from their entry.
-  Do not use close to cellular telephones (maintain at least one meter of distance).
-  To operate close (less than 1 meter) to an instrument with short waves therapy or microwaves, because it can produce instability in the emission.
-  Don't simultaneously connect the patient with the TEKRA LIFE and a surgical instrument HF, to avoid dangers for the patient and for the instrument.
-  The device works according to its specifications if the environment is maintained to a temperature between the 5° and the 40° C and with damp inferior to the 80%. The same conditions must be maintained during the transport and the store.
-  In case of malfunctions and breakdowns, it is opportune to exclusively send the device to the mother company.
-  It's recommended not to operate in proximity of inflammable substances.
-  Don't use gel and different accessories from those furnished in endowment.
-  It's important to inform the patient on the type of feeling to perceive during the therapy, to immediately intervene or interrupt the session switching off the device or removing the solenoids, in case of different or uncorrect perceptions.
-  If the power or its regulation cause elevated heating or pain in the treated zone it is necessary to immediately reduce the intensity of stimulation or eventually interrupt the application.
-  Keep away from children.

9

TECHNICAL FEATURES

9.1 - Power supply

PRI: 230V ~ 50Hz SEC: 12V - 1,4A

9.2 - Output features

Max power intensity: 30W

Frequency (f): 500 kHz

9.3 - Other features

> **Dimensions:** 175x105x40h [mm]

> **Weight:** 400 [g]

> **Class:** II^o

> **Type:** BF

> **Classification to liquids entry:** IP20

> **Safety in presence of inflammable anaesthetic gas:** it's not of AP or APG cathegory

> **Device for working:** continuous


Built following rules

EN 60601-1 (2007) – Electromedical devices: general safety rules

EN 60601-1-2 (2003) – Collateral Rules: Electro-magnetic compatibility – Prescriptions and tests

EN 60601-1-4 (1997): Collateral Rule: Programmable electromedical devices

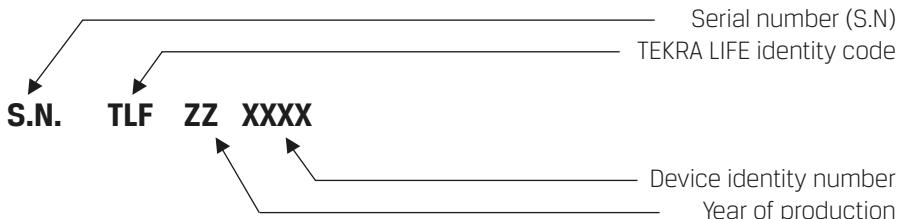EN 980-2003 en EN 1041 – symbols for electromedical devices.

10

SYMBOLS

TYPE BF DEVICE

ATTENTION, READ THE ENCLOSED DOCUMENTS

THIS DEVICE IS MARKED ACCORDING TO DIRECTIVE CEE 93/42 AND 2007/47/CE.

11

ENDOWMENT OF BASE / ACCESSORIES

11.1 - Basic Endowment

Device

Feeder

N.1 headpiece

Instructions

Carrying case

11.2 - Accessories and consumed materials

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NewAge Italia Srl - Via De Brozzi, 3 - Lugo (RA) 48022 - Tel. +39.0545.32019