


EXPERTS IN PHYSIOTHERAPY

Massage, Training & Spa



SINCE 1904

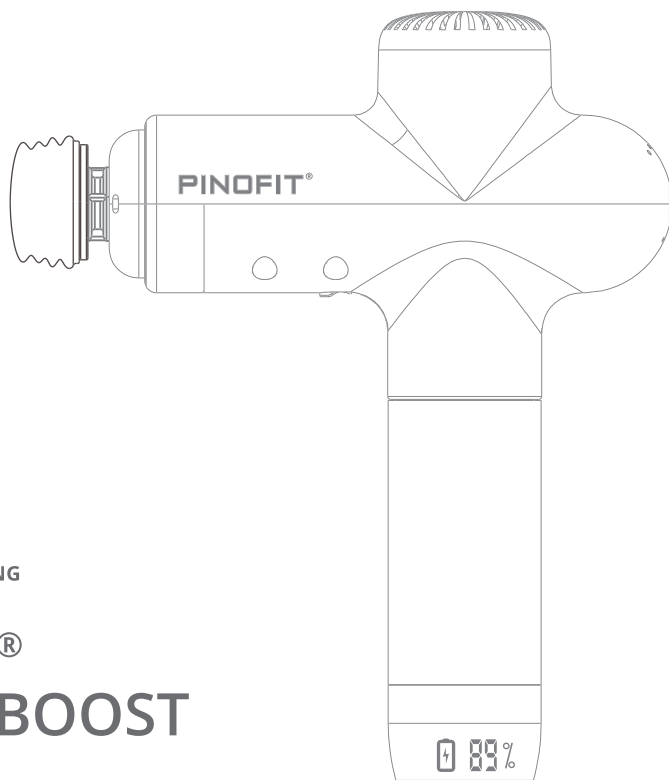

BEDIENUNGSANLEITUNG


PINOFIT® PHYSIO BOOST PRO

- UK INSTRUCTIONS FOR USE
NL HANDLEIDING
FR MODE D'EMPLOI
IT ISTRUZIONI PER L'USO
ES INSTRUCCIONES DE SERVICIO
PT MANUAL DE UTILIZAÇÃO
AR مَادَخْتَسَالَا تَامِيْلَعَت

PINOFIT® PHYSIO BOOST PRO

OVERVIEW	11
MESSAGE ATTACHMENTS	12
TECHNICAL SPECIFICATIONS & FEATURES	12
THE MODERN MESSAGE DEVICE.....	13
IMPORTANT SAFETY INFORMATION.....	13
INSTRUCTIONS FOR USE	15
1. Charging the device.....	15
2. Attaching the massage attachments.....	15
3. Using the multifunctional button	15
4. Removing the battery	16
Optional Accessoires	14
Notes	16
Cleaning and caring for the device	17
Maintenance.....	17
Disposal.....	17
Disclaimer	17

PINOFIT® PHYSIO BOOST PRO

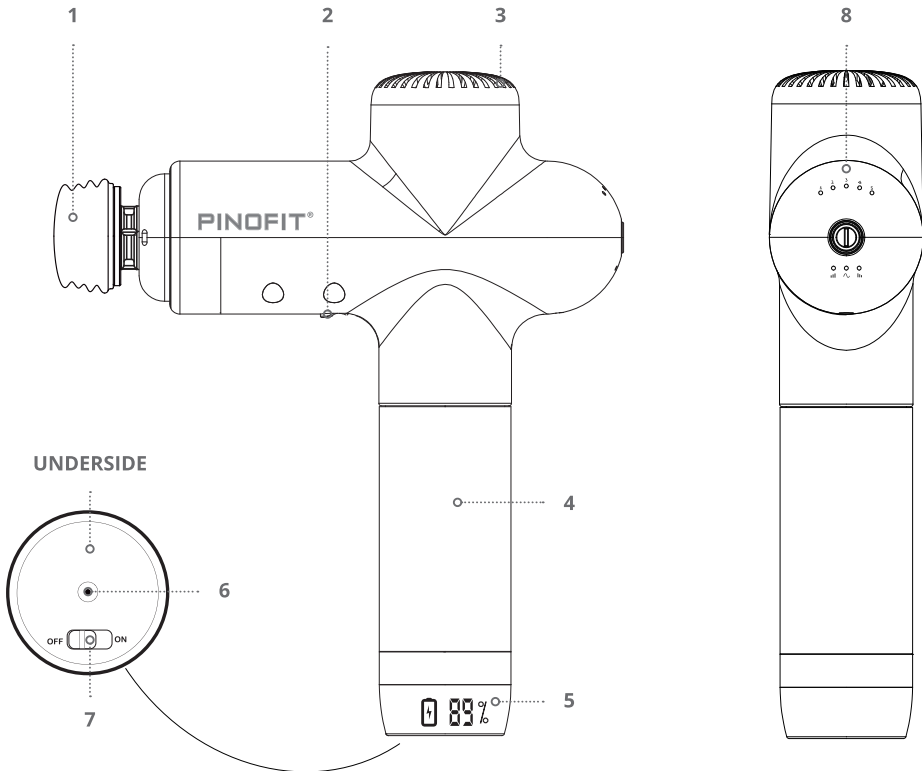

1. Quick-release massage attachment
2. Battery removal switch
3. Control light

4. Silicone handle
5. Digital battery charging level indicator
6. Charging interface

7. ON/OFF switch
8. Control panel with multifunctional button



CE marking



Only use in dry indoor areas.



Device with protection class II (Adapter: AC input and DC output are safely electrically isolated)




Battery



Hazardous goods must be disposed of as special waste.


Electronic parts may not be disposed of as household waste.

MASSAGE ATTACHMENTS

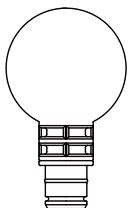


Large, round attachment

Full-body massage and large muscle groups

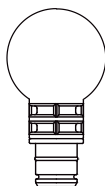

Small, round attachment

Full-body massage and small muscle groups

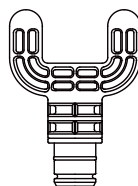

Forked attachment with removable silicone cushions

Muscle massage



Air cushion attachment with silicone cushion

Massage of sensitive parts of the body, full-body massage



Flat attachment

Full-body massage
Stimulating massage



Bullet attachment with removable silicone cushion

Muscle and trigger-point massage

TECHNICAL SPECIFICATIONS & FEATURES

PHYSIO BOOST PRO

Rated speed: 1,300 – 3,300 rpm

Five frequency modes, three automatic modes

Max. output: 126 W

Max. operating time: approx. 4 h

Volume: approx. 60 dB

Weight: approx. 1,010 g

Protection class II

Dimensions: approx. 260 x 190 x 60 mm

BATTERY

Rated voltage: 24 V DC

Type: Li-Ion

Capacity: 2,400 mAh

Max. charging time: approx. 3 h

CHARGER

Rated input power: 100–240V ~ 50/60 Hz

Rated output power: 24 V/1A

Do not use the device for more than 10 minutes at a time! Please read the operating instructions thoroughly and carefully as they contain important information about using the device. Hold on to the instructions for future reference.

THE MODERN MASSAGE DEVICE

The PINOFIT® Physio Boost Pro is an innovative vibrating massage device for use in wellness massage. **Not a medical product** – Not approved for use as part of physical therapy or prescribed medical treatments.



IMPORTANT SAFETY INFORMATION:

The following basic safety information serves to prevent personal injury and material damage:

- Before connecting the device to the power supply, make sure that the mains voltage is compatible with the supply voltage indicated on the nameplate.
- The device may only be used with the supplied, certified charger.
- Do not disassemble or modify the device.
- Do not remove the battery while the Physio Boost Pro is switched on.
- The device is designed for use in dry indoor spaces only.
- Keep the device away from heat sources, liquids and children.
- The device may only be used by qualified professionals (e.g. physiotherapists, spa workers, masseuses), and is not intended for use by the end consumer.
- The instructions in this manual are not a substitute for medical advice.
- The Physio Boost Pro is designed to support wellness massages. It is not suitable for any other use.
- The device should be used over clothing on a clean, dry surface of the body by applying light pressure and moving it across the body for approx. 60 seconds per area.
- Only use the supplied attachments to achieve the desired effect. The use of accessories that are not expressly recommended by the manufacturer may result in injury to the user and damage to the device. For this reason, only original accessories should be used.
- Pressure points may occur regardless of the control settings or the applied pressure. Inspect the treated areas regularly and discontinue use immediately at the first sign or pain or discomfort.
- To prevent injury by crushing, keep fingers, hair and other body parts away from the spindle and the back of the applicator.



Do not touch the front part of the Physio Boost while the device is running without the massage attachments installed.



Do not touch the front part between the Physio Boost and the massage attachment while the device is running.

- The PINOFIT® Physio Boost Pro is not designed for continuous use. Do not use the device for longer than 10 minutes. After a 10-minute session with the Physio Boost Pro, wait at least five minutes before continuing use.
- Do not use the device on the head, or near the genitals or orifices.
- Improper use/handling of the device may cause harm to health. The user does so at their own risk.
- Make sure that the ventilation openings of the device are free from dust and dirt.
- Check the Physio Boost Pro and the massage attachments for any damage before each use (e.g. sharp edges).
- **Not a toy** – Keep out of reach of children: Children under the age of 18 must not use the Physio Boost Pro unsupervised.
- Keep the packaging materials away from children. They pose a risk of suffocation.

INSTRUCTIONS FOR USE

Remove all packaging materials before first use.

1. CHARGING THE DEVICE

- Charge the device before first use.
- Make sure that the Physio Boost Pro is switched off.
- Connect the charger. The charging indicators will flash in an alternating pattern to indicate that the device is charging.
- When the battery display reads 100%, the battery is fully charged.
- Do not move the device while it is charging.
- Disconnect the charger from the device after charging or before use.

2. ATTACHING THE MESSAGE ATTACHMENTS

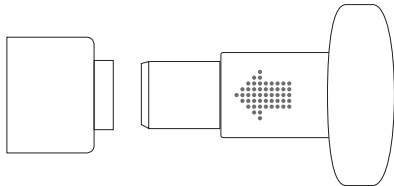


Figure 1: Installation

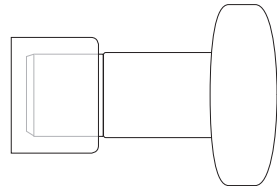


Figure 2: Result

- Switch off the Physio Boost Pro and insert the desired massage attachment into the opening until it reaches the stop.
- The result is shown in Fig. 2.
- Massage the desired body part at the desired speed, and apply the necessary amount of pressure.
- After switching off the device, the massage attachment can be manually pulled off and exchanged.


3. USING THE MULTIFUNCTIONAL BUTTON

- Switch on the device using the ON/OFF switch on the underside. The current charging status appears on the battery display and the upper part of the device lights up.
- Hold the multifunctional button for two seconds to start the device or to switch between standard and automatic mode.
 - a) In standard mode: briefly press the multifunctional button to start the device on level 1. The level increases each time the button is pressed. After level 5, press the button again to switch off the device. If you press the multifunctional button and hold it for two seconds, the device switches to automatic mode.

b) In automatic mode: briefly press the multifunctional button to start the device with diminishing vibrations. Press the multifunctional button again to cycle through the additional automatic modes. Press the multifunctional button and hold it for two seconds to switch to standard mode.

- The device switches off automatically every 10 minutes. Press the multifunctional button again to continue using the Physio Boost Pro.

4. REMOVING THE BATTERY

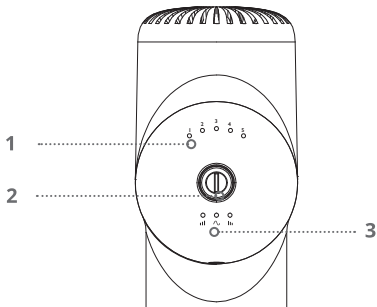

1. Standard mode

1–5 intensity levels available

2. Multifunctional button

3. Automatic mode:

▮ Diminishing vibrations

Diminishing intensity of vibrations, e.g. for full-body massage after exercise


▮ Increasing vibrations

Increasing intensity of vibrations, e.g. for full-body massage before exercise.

~ Oscillating vibrations

Increasing and diminishing intensity of vibrations, e.g. for relaxation

- Switch off the device before removing the battery.
- Slide the battery removal switch towards the massage attachment with one hand and keep it tightly pressed.
- Hold the handle with the other hand and slowly pull out the battery.

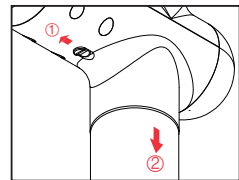

OPTIONAL ACCESSORIES: CHARGING STATION

Connect the charger to the charging station. When you insert the mains plug into a power socket, the charging station starts to light up. Place the switched off Physio Boost Pro vertically in the charging station. If it is correctly positioned in the charging station, the charging station will start to flash during charging. The charging process is complete when the charging station is constantly lit.

Please note: The charging station is available separately and is not included in the PINOFIT® Physio Boost Pro delivery.

NOTES

If you notice a white residue on the shaft of the massage attachments, this can easily be cleaned off with a damp cloth and a small amount of washing up liquid. This residue is not caused by a quality defect, but a technically necessary lubricant that is used in the device.

CLEANING AND CARING FOR THE DEVICE

- If you are not going to use the device for a longer period, switch it off and remove the battery. Store it in a cool, dry and secure location.
- Charge the device every six months while it is not in use.
- Use a neutral cleaning product and a soft cloth for daily cleaning. Do not use engine oil, organic solvents or other chemicals to clean the device.
- Switch off the device after use or before cleaning.
- The device must be firmly secured during transport. Avoid heavy vibrations, impacts, tumbles and falls.

MAINTENANCE

- Do not open the battery of the Physio Boost Pro.
- If the device does not function correctly, do not try to repair it or correct the fault by disassembling the device yourself. Doing so will automatically void the warranty. Repairs may only be carried out by an approved electrical technician.

DISPOSAL

In the event of disposal, PINO GmbH is able to dispose of the device for you. To do so, the device must be returned postage paid to PINO GmbH with a clear request for its disposal. In accordance with the German Electrical and Electronic Equipment Act (ElektroG) and EU Directive 2012/19/EU (WEEE Directive), the device may not be disposed of at public waste collection points.

DISCLAIMER

We cannot accept any liability for damage caused by improper or incorrect use of the device.

Like us on Facebook!

<https://www.facebook.com/pinoshop.de/>

Follow us on YouTube!

<https://www.youtube.com/pinofit>

Vertrieb | Distribution | Sales | Ventes | Vendite | Venta | De vendas
PINO Pharmazeutische Präparate GmbH
Paul-Dessau-Str. 5 | 22761 Hamburg/Germany

service@pino-gmbh.de



08/2020